


**Note:**

When using only one Wrap, place the Control Unit on the opposite side of the injured hock and place the Single Connector Hose over the back of the horse, attach the Single Connector Hose to the Wrap. Connect the opposite end of the Connector Hose into the Control Unit.

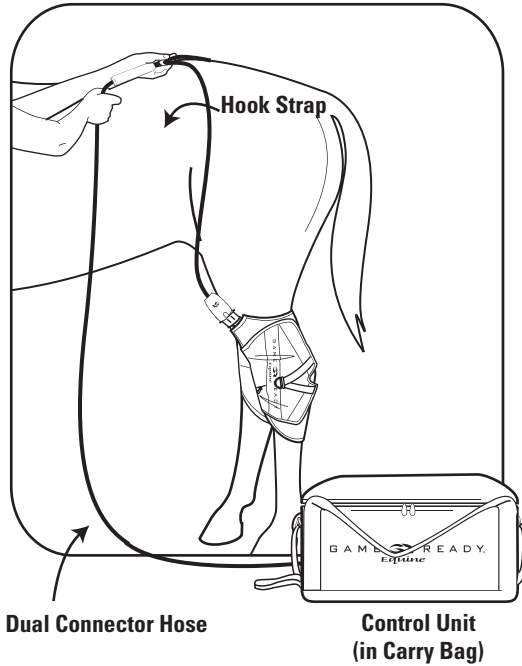

When using two Wraps simultaneously, use the Dual Connector Hose. Raise the Hose up and over the hindquarters of the horse to make a "Y" split. Connect each Hose to its corresponding Wrap. Adjust the length of the Hose by moving the Hook Strap up or down. Connect the single opposite end of the Connector Hose into the Control Unit.

Do not allow the Hose to bend the Wrap at the connection. This can restrict water flow and affect cooling.


After treatment, disconnect the Hose from the Wrap first, before removal of the Wrap from the horse.

**Care:** Gently remove the Heat Exchanger from the Sleeve. Hand or machine wash the Sleeve in cold water and mild detergent, or antibacterial soap. Hang to dry. Hand wash the Heat Exchanger, do not machine wash; doing so will void the warranty. Periodically run hydrogen peroxide through your Game Ready Equine System to keep the interior water chamber of your Wrap clean. See Control Unit User's Manual for more detail. Cold sterilization only.


**Warranty Information:**

**Sleeve:** In case of manufacturer defect: Sleeve may be returned within 7 days of purchase.

**Heat Exchanger:** 1 year from date of purchase. See warranty card included with the Heat Exchanger.

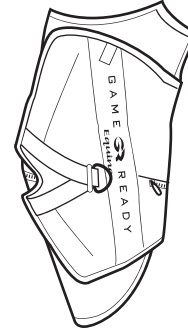


Right Hock Sleeve: 540322  
Right Hock Heat Exchanger: 560322

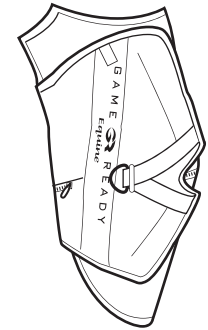

Left Hock Sleeve: 540324  
Left Hock Heat Exchanger: 560324

**IMPORTANT SAFETY INFORMATION**

- 1 Always consult with your veterinarian before using the Game Ready Equine System for an injury.
- 2 During treatment, the horse must be accompanied by an attendant.
- 3 Do not place the Wrap directly against open wounds, sores, rashes, infections, or stitches. First place a sterile dressing on the wound and then apply the Wrap.
- 4 Do not store in a closed plastic bag after use, as this may promote mold growth.
- 5 Do not puncture the Wrap; doing so will void the Game Ready Equine warranty.
- 6 The first application of this Wrap will likely cause most horses to flex their hind leg. The Quick-Release Strap is provided for your safety, so that the Wrap can be removed quickly until the horse is familiar with the sensation. Once the individual horse is familiar with the Wrap, the Quick-Release Strap may be removed.
- 7 It is **recommended** that the highest level of pressure is Medium Pressure (—▲▲▲) for this Wrap. If the pressure is uncomfortable, pause the Unit, and restart with lower pressure.

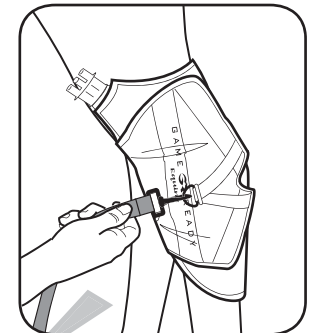


**QUICK-RELEASE STRAP INCLUDED**

## Hock Wrap =


### Sleeve + Heat Exchanger

1

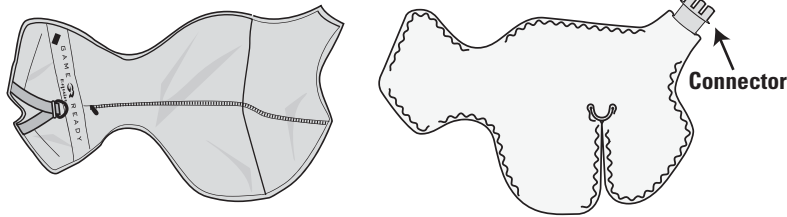


**fig. 1** Place the Sleeve on a flat surface with the logo side up and open the zipper.

2

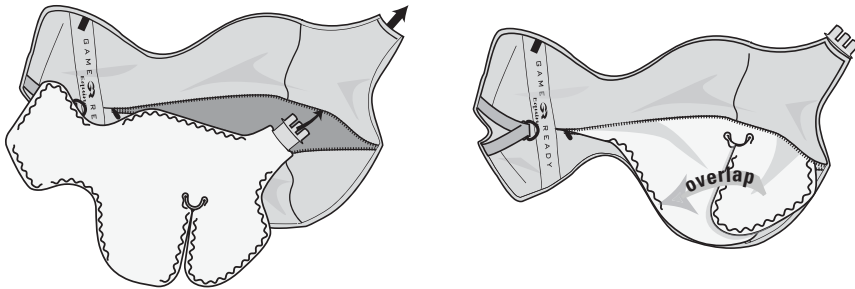

**fig. 2** Insert the Heat Exchanger into the Sleeve with the green side down. Slide the Connector through the opening in the Sleeve. The flaps of the Heat Exchanger must overlap slightly to fit into the Sleeve. The green side of the Heat Exchanger should touch the green side of the Sleeve.

3

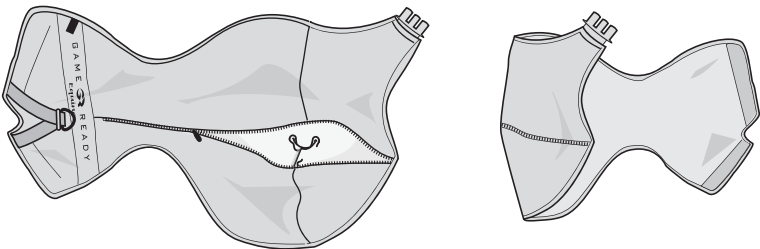


**fig. 3** Make sure the Heat Exchanger is flat inside the Sleeve with no folds or creases. Zip up the Sleeve. After assembly, fold at the natural vertical fold, and make sure the Wrap is flat and smooth.

## Hock Wrap Heat Exchanger Removal

- 1 Disconnect the Connector Hose from the Wrap
- 2 Unzip the zipper
- 3 Gently pull out the Heat Exchanger


## Hock Wrap Application

(left leg illustrated)

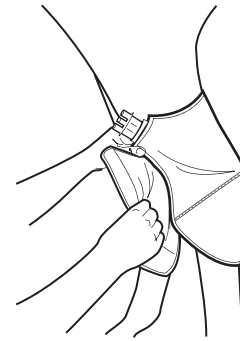


**Note:** For maximum cooling, place the Wrap against the horse's bare leg, unless there is a wound. In the case of a wound, cover with a bandage before applying the Wrap.

**Step 1:** Place the Wrap around the horse's leg, aligning the Hose connection on the outside of the leg.

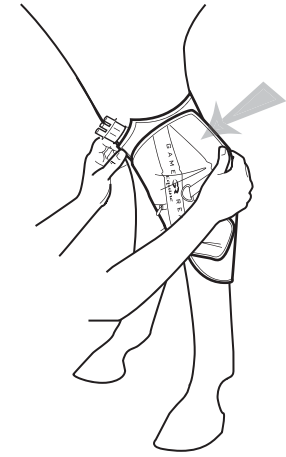

**Step 2:** Hold the Wrap above the hock and secure the top flap around the leg so that the Wrap fits snugly around the horse's leg, above the hock joint.

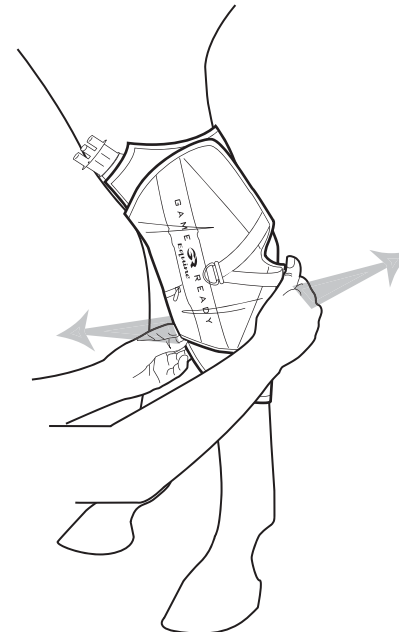

**Step 3:** Pull the lower inside edge so that the Wrap is snug against the leg below the joint. Secure with the lower flap.

**Caution:** Do not open the zipper on the Wrap while the Wrap is in use, or while being applied to the horse.

**Protocol:** 30-40 minute application on low, medium or no pressure. Always consult your veterinarian before using the Game Ready Equine System on any injury.