



Large Utility Wrap Application

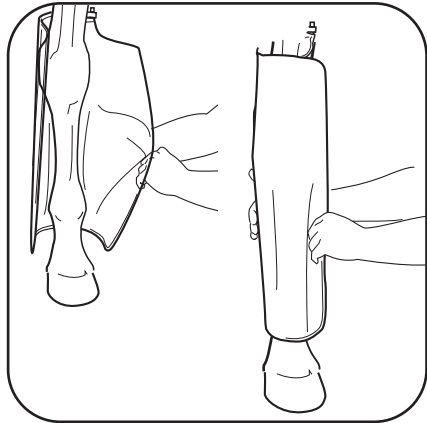

Place the Wrap(s) around the horse's leg(s). The Large Utility Wrap can be applied on the leg wherever swelling may occur. Secure by adjusting the hook and loop for a snug fit. The Wrap connector should align vertically on the outside of the horse's leg.

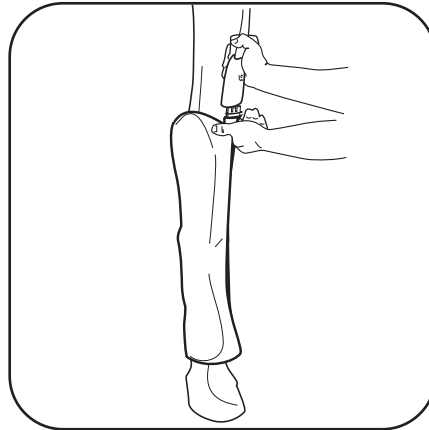



*When using one Wrap, attach the Single Connector Hose to the Wrap. Connect the opposite end of the Connector Hose into the Control Unit.

**When using two Wraps simultaneously, use the Dual Connector Hose. Raise the hose up and over the withers of the horse to make a "Y" split. Connect each hose to its corresponding Wrap. Adjust the length of the hose by moving the barrel lock up or down. Connect the single opposite end of the connector hose into the Control Unit. Do not allow hose to bend the Wrap at connection. This can impede flow, and affect cooling.

During treatment, the horse must be accompanied by an attendant.

After treatment, disconnect the Hose from the Wrap first, before removal of Wrap from the horse.

Caution: Do not open zipper on the Wrap while Wrap is in use, or while being applied to the horse.

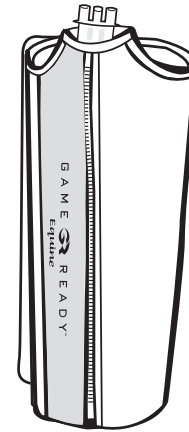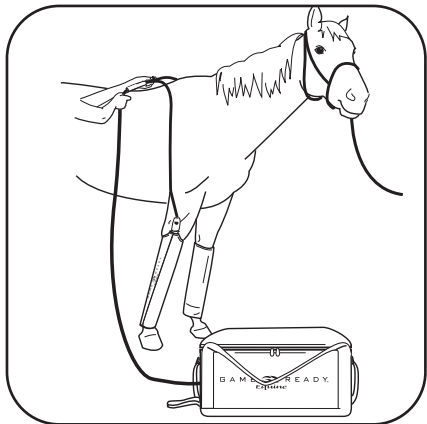

Large Utility Sleeve: 540130
Large Utility Heat Exchanger: 560130

Warning: Always consult with your veterinarian before using the Game Ready Equine System for an injury.

Note: For maximum cooling, place Wrap against bare leg.

Caution: Do not place directly against open wounds, sores, rashes, infections, or stitches. First place a sterile dressing on the wound and then apply the Wrap. If pressure is uncomfortable, pause unit, and restart with lower pressure. Do not store in closed plastic bag after use, as this will promote mold growth. Do not puncture Wrap, doing so will void Game Ready Equine warranty.

Care: Gently remove Heat Exchanger from the Sleeve. Hand wash the Sleeve in cold water and mild detergent, or antibacterial soap. Hang to dry. Hand wash the Heat Exchanger, **do not machine wash**. Doing so will void warranty. Periodically run hydrogen peroxide through your Game Ready Equine System to keep the interior water chamber of your Wrap clean. See Control Unit User's Manual for more detail. Cold sterilization only.

Warranty Information:

Sleeve: In case of manufacturer defect, sleeve may be returned within 7 days of purchase.

Heat Exchanger: 1 year from date of purchase. See warranty card included with Heat Exchanger.

CoolSystems, Inc.
1201 Marina Village Pkwy Suite 200
Alameda, CA 94501
1-888-GAMEREDY



Large Utility Wrap =

Large Utility Sleeve


+

Large Utility Heat Exchanger


1

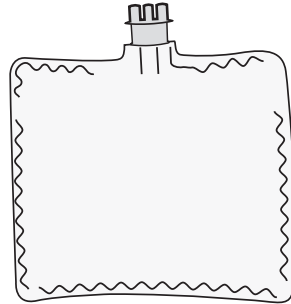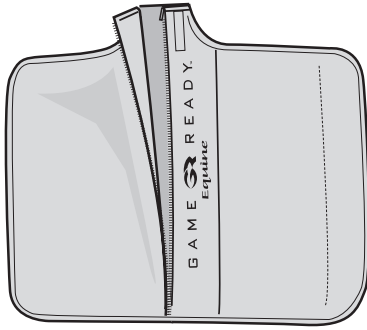

2

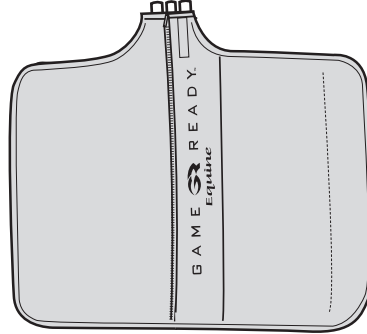

3

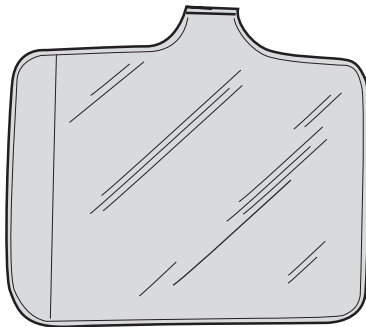

Large Utility Wrap (Sleeve/Heat Exchanger) - Insertion

fig. 1 Place Sleeve on flat surface with logo side up and open the zipper

fig. 2 Insert Heat Exchanger into the Sleeve with blue side down
(Blue side of Heat Exchanger should touch olive side of Sleeve)

fig. 3 Make sure the Heat Exchanger is flat inside the Sleeve with no folds or creases. Zip up the Sleeve. After assembly, lay the olive side up, make sure the Wrap is flat and smooth

Large Utility Wrap (Sleeve/Heat Exchanger) - Removal

- 1 Disconnect the Connector Hose from the Wrap
- 2 Unzip the zipper
- 3 Gently pull out the Heat Exchanger