

Back Sleeve: 510500  
Back Heat Exchanger: 520500

Thank you for choosing the Game Ready System. To assure you have the best possible experience please be sure to read the entire User's Manual prior to the use of this product.

**Warnings**

- Always consult with your physician before using the Game Ready System for any injury. Your physician can determine whether or not the System is appropriate for your injury, and can provide advice on how to best use the System to treat your injury.
- If used incorrectly, cold therapy can cause frostbite or other tissue damage. Always follow the protocol provided by your physician.
- Do not place directly against open wounds, sores, rashes, infections, or stitches. May be applied over clothing or dressing.
- If use of the Game Ready System causes increased pain, swelling, discoloration or loss of sensation, discontinue use and consult with your physician.
- If pressure is uncomfortable, pause unit, and restart with lower pressure.

**Care:** Gently remove Heat Exchanger from the Sleeve. Hand or machine wash the Sleeve in cold water and mild detergent, or antibacterial soap. Hang to dry. Hand wash the Heat Exchanger, do not machine wash.

**Warranty Information:**

*Sleeve:* In case of manufacturer defect, Sleeve may be returned within 7 days of purchase.

*Heat Exchanger:* 1 year from date of purchase. See warranty card included with Heat Exchanger.

CoolSystems, Inc.

1201 Marina Village Pkwy Suite 200


Alameda, CA 94501

1.888.GAMEREDY




EC REP  
EMERGO EUROPE  
Molenstraat 15  
2513 BH, The Hague  
The Netherlands


**Back Wrap Application**

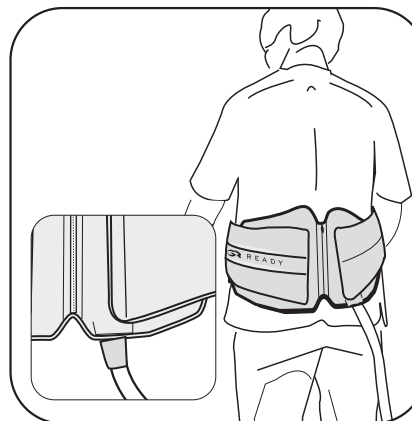


Connect the Hose to the Wrap first then secure it snugly around the back or attach the Wrap first if someone else is helping to apply it.

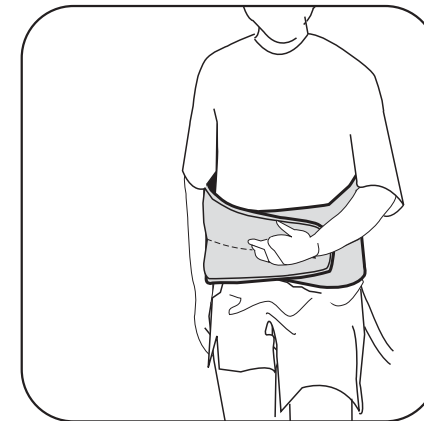

With blue side facing your back, wrap belt around your waist and secure belt ends. Once treatment ends, remove the Wrap from the back then disconnect the Hose from the Wrap, to avoid dripping water.

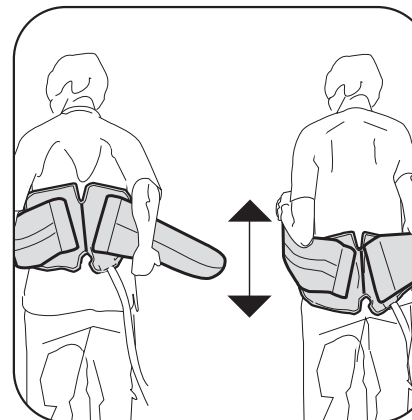

The Back Wrap is designed to be useful for both upper and lower back treatment, and should fit a wide range of body-types.

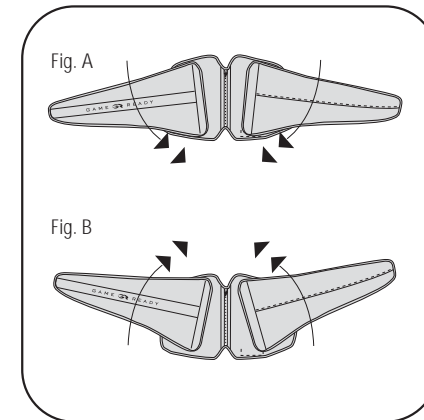

Experiment with the positioning of the belt wings to get the best fit for your treatment area. To effectively treat the ribcage or upper back, angle belt wings down (Figure A). To effectively treat the lower back, angle belt wings up (Figure B).





## Back Wrap =

Back Sleeve



+

Back Heat Exchanger


1

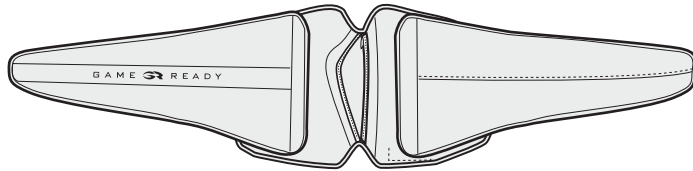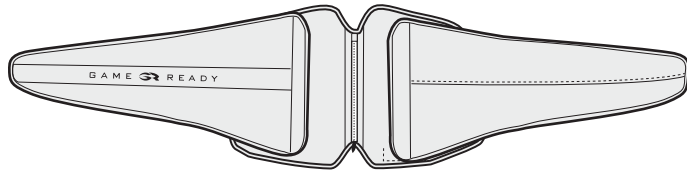

2

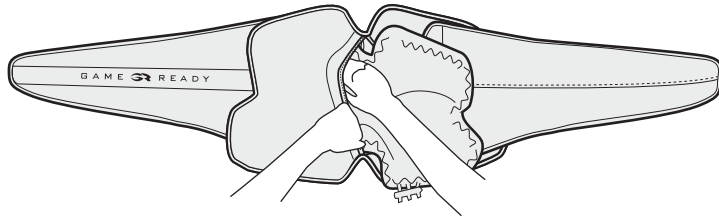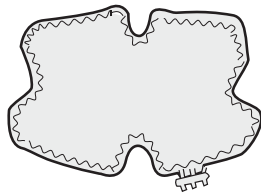

3

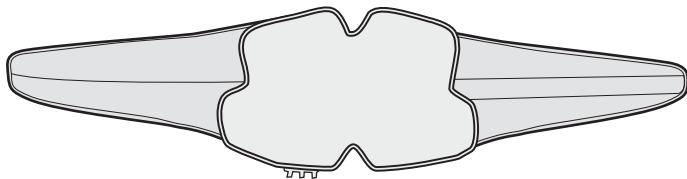

GAME READY®

Back Wrap #590500

### Back Wrap (Sleeve/Heat Exchanger) - Insertion

**fig. 1** Place Sleeve on flat surface with logo side up and open the zipper

**fig. 2** Insert Heat Exchanger into the Sleeve with blue side down (Blue side of Heat Exchanger should match blue side of Sleeve)

**fig. 3** Make sure the Heat Exchanger is flat inside the Sleeve with no folds or creases. Zip up the Sleeve. After assembly, lay the blue side up, make sure the Wrap is flat and smooth

### Back Wrap (Sleeve/ Heat Exchanger) - Removal

- 1 Disconnect the Connector Hose from the Wrap
- 2 Unzip the zipper
- 3 Gently pull out the Heat Exchanger