



Canine Stifle Wrap Application

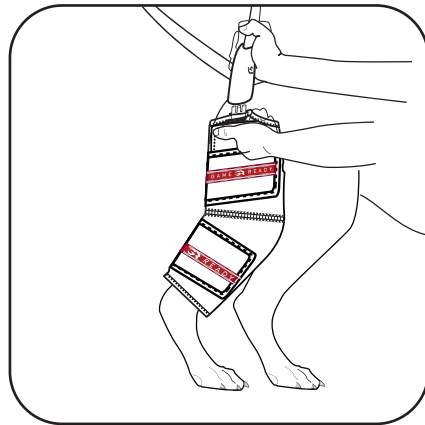

Place the Stifle Wrap(s) around the dog's hind leg(s). Position the Wrap on the dog's stifle, matching the bend in the Wrap to the bend in the dog's leg. Secure by using the Wrap Straps and adjust as necessary to create a comfortable, snug fit. The Wrap connector should align vertically on the outside of the dog's leg.

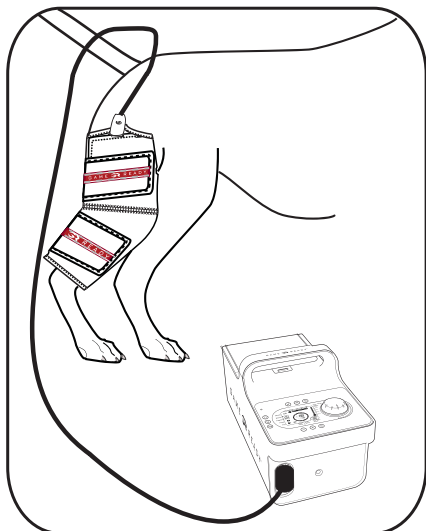


*When using one Wrap, attach the Single Connector Hose to the Wrap. Connect the opposite end of the Connector Hose into the Control Unit.

**When using two Wraps simultaneously, use the Dual Connector Hose. Raise the hose up and over the hind end of the dog to make a "Y" split. Connect each hose end to its corresponding Wrap. Adjust the length of the hose by moving the barrel lock up or down. Connect the single opposite end of the Connector Hose into the Control Unit. Do not allow hose to bend the Wrap at connection. This can impede flow, and affect cooling.

During treatment, the dog must be accompanied by an attendant.

After treatment, disconnect the Hose from the Wrap first, before removal of Wrap from the dog.

Caution: Do not open zipper on the Wrap while Wrap is in use.

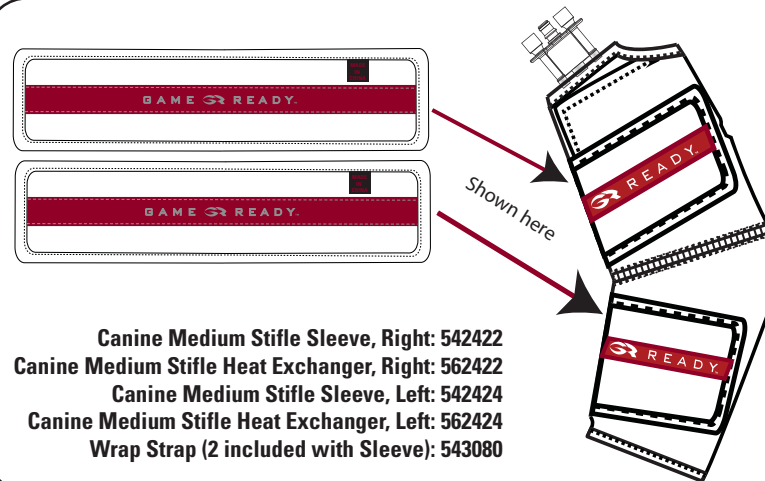

Canine Medium Stifle Sleeve, Right: 542422
 Canine Medium Stifle Heat Exchanger, Right: 562422
 Canine Medium Stifle Sleeve, Left: 542424
 Canine Medium Stifle Heat Exchanger, Left: 562424
 Wrap Strap (2 included with Sleeve): 543080

Warning: Always consult with your veterinarian before using the Game Ready System.

Caution: Do not place directly against open wounds, sores, rashes, infections, or stitches. First place a sterile dressing on the wound and then apply the Wrap. If pressure appears uncomfortable, pause treatment, and restart with lower pressure. Do not store in closed plastic bag after use, as this will promote mold growth. Do not puncture Wrap, doing so will void Game Ready warranty.

Care: Gently remove Heat Exchanger from the Sleeve. Hand wash the Sleeve in cold water and a mild detergent, or antibacterial soap. Hang to dry. Wipe down the Heat Exchanger with cold water and a mild detergent, **do not machine wash**. Doing so will void warranty. Periodically run hydrogen peroxide through your Game Ready System to keep the interior water chamber of your Wrap clean. See Control Unit User's Manual for more detail.

Warranty Information:

Sleeve: In case of manufacturer defect, sleeve may be returned within 7 days of purchase.

Heat Exchanger: 1 year from date of purchase. See warranty card included with Heat Exchanger.

CoolSystems, Inc.
 1201 Marina Village Pkwy Suite 200
 Alameda, CA 94501
 1-888-GAMEREADY
 www.gamereadycanine.com

Canine Stifle Wrap =

Stifle Heat Exchanger

+

Stifle Sleeve

Fig 1

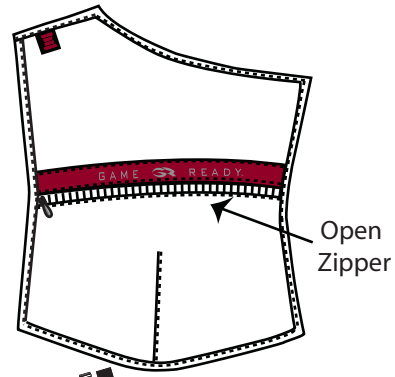
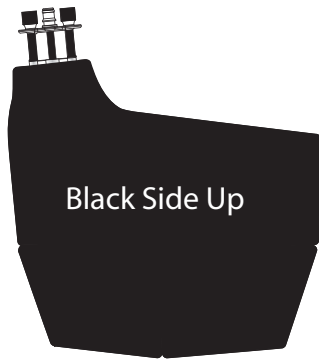


Fig 2

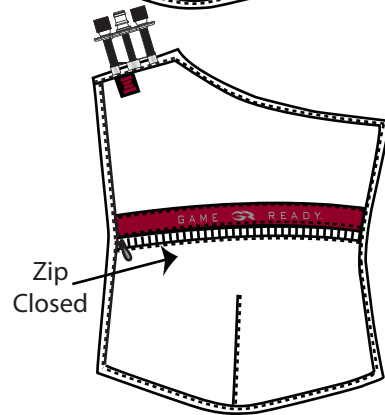
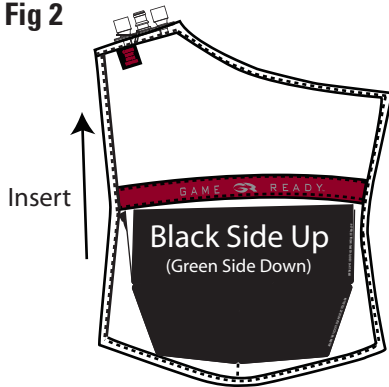


Fig 3


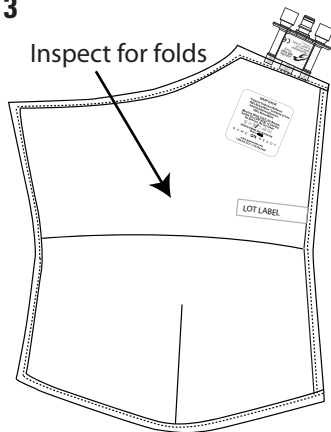


Fig 4 Attaching Straps

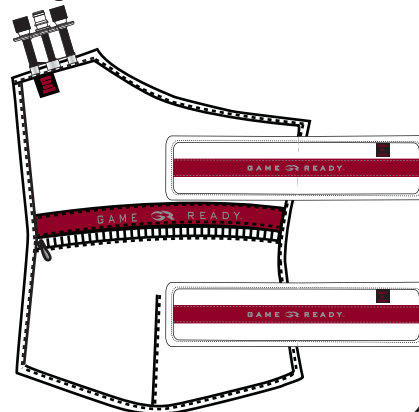

GAME READY™

Canine Stifle Wrap

Stifle Wrap (Sleeve/Heat Exchanger) - Assembly

- fig. 1** Organize Heat Exchanger with Sleeve on flat surface. Place Sleeve with logo side up and open the zipper.
- fig. 2** Insert Heat Exchanger into the Sleeve with green side down. (Green side of Heat Exchanger should touch green side of Sleeve) Make sure the Heat Exchanger is flat inside the Sleeve with no folds or creases. Zip up the Sleeve.
- fig. 3** After assembly, lay the green side up, make sure the Wrap is flat and smooth.
- fig. 4** To apply straps, attach one end of the hook portion of each strap to one side of the Stifle Sleeve as shown.

Stifle Wrap (Sleeve/Heat Exchanger) - Disassembly

- 1 Disconnect the Connector Hose from the Wrap
- 2 Unzip the zipper
- 3 Gently pull out the Heat Exchanger