



Canine Utility Wrap Application

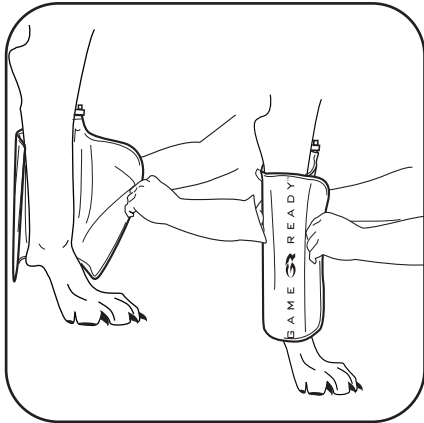

Place the Wrap(s) around the dog's leg(s). The Utility Wrap can be applied on the leg wherever swelling may occur. Secure by adjusting the hook and loop closure for a snug fit. The Wrap connector should align vertically on the outside of the dog's leg.

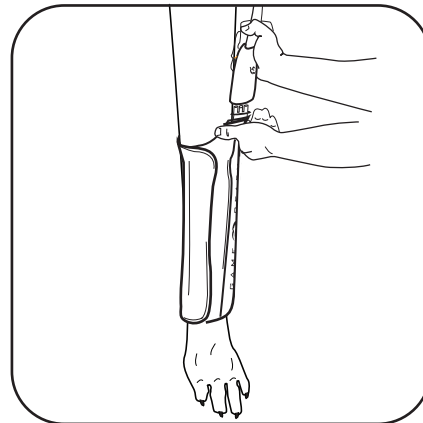

*When using one Wrap, attach the Single Connector Hose to the Wrap. Connect the opposite end of the Connector Hose into the Control Unit.

**When using two Wraps simultaneously, use the Dual Connector Hose. Raise the hose up and over the shoulders of the dog to make a "Y" split. Connect each hose end to its corresponding Wrap. Adjust the length of the hose by moving the barrel lock up or down. Connect the single opposite end of the Connector Hose into the Control Unit. Do not allow hose to bend the Wrap at connection. This can impede flow, and affect cooling.

During treatment, the dog must be accompanied by an attendant.

After treatment, disconnect the Hose from the Wrap first, before removal of Wrap from the dog.

Caution: Do not open zipper on the Wrap while Wrap is in use, or while being applied to the dog.



Utility Sleeve: 542100
Utility Heat Exchanger: 562100

Warning: Always consult with your veterinarian before using the Game Ready System.

Caution: Do not place directly against open wounds, sores, rashes, infections, or stitches. First place a sterile dressing on the wound and then apply the Wrap. If pressure appears uncomfortable, pause treatment, and restart with lower pressure. Do not store in closed plastic bag after use, as this will promote mold growth. Do not puncture Wrap, doing so will void Game Ready warranty.

Care: Gently remove Heat Exchanger from the Sleeve. Hand wash the Sleeve in cold water and mild detergent, or antibacterial soap. Hang to dry. Hand wash the Heat Exchanger, **do not machine wash**. Doing so will void warranty. Periodically run hydrogen peroxide through your Game Ready System to keep the interior water chamber of your Wrap clean. See Control Unit User's Manual for more detail. Cold sterilization only.

Warranty Information:

Sleeve: In case of manufacturer defect, sleeve may be returned within 7 days of purchase.

Heat Exchanger: 1 year from date of purchase. See warranty card included with Heat Exchanger.

CoolSystems, Inc.
1201 Marina Village Pkwy Suite 200
Alameda, CA 94501
1-888-GAMEREADY
www.gamereadycanine.com

Canine Utility Wrap =

Utility Sleeve

+

Utility Heat Exchanger

Fig 1

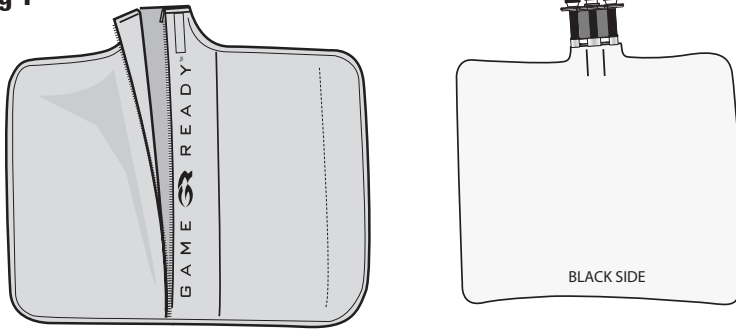


Fig 2



Fig 3



GAME READY™

Canine Utility Wrap

Utility Wrap (Sleeve/Heat Exchanger) - Insertion

fig. 1 Place Sleeve on flat surface with logo side up and open the zipper

fig. 2 Insert Heat Exchanger into the Sleeve with green side down
(Green side of Heat Exchanger should touch green side of Sleeve)

fig. 3 Make sure the Heat Exchanger is flat inside the Sleeve with no folds or creases. Zip up the Sleeve. After assembly, lay the green side up, make sure the Wrap is flat and smooth

Utility Wrap (Sleeve/Heat Exchanger) - Removal

- 1 Disconnect the Connector Hose from the Wrap
- 2 Unzip the zipper
- 3 Gently pull out the Heat Exchanger