



Full Leg Wrap Application

(left leg illustrated)

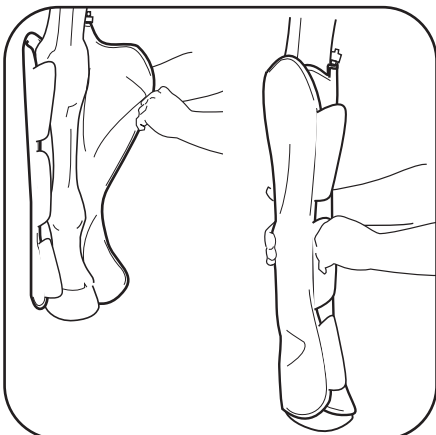

Place the Wrap around the horse's leg, aligning Hose Connection on the outside of the leg. Hold the Wrap behind the knee and secure the hook and loop flaps in front of the leg so that the Wrap fits snugly around the horse's leg, from the top of the knee down to the hoof.

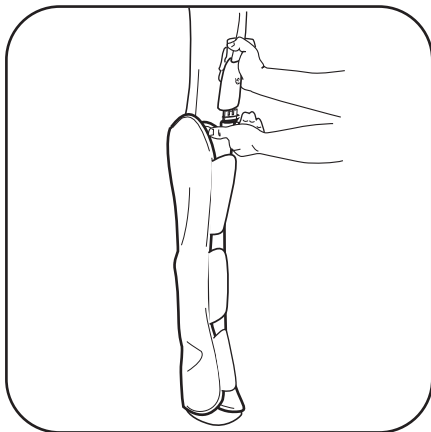

*When using one wrap, attach the Single Connector Hose to the Wrap. Connect the opposite end of the Connector Hose into the Control Unit.



**When using two Wraps simultaneously, use the Dual Connector Hose. Raise the Hose up and over the withers of the horse to make a "Y" split. Connect each Hose to its corresponding Wrap. Adjust the length of the Hose by moving the barrel lock up or down. Connect the single opposite end of the Connector Hose into the Control Unit.

Do not allow Hose to bend the wrap at connection. This can impede flow, and affect cooling.


During treatment, the horse must be accompanied by an attendant.

After treatment, disconnect the Hose from the Wrap first, before removal of Wrap from the horse.

Caution: Do not open zipper on the Wrap while Wrap is in use, or while being applied to the horse.

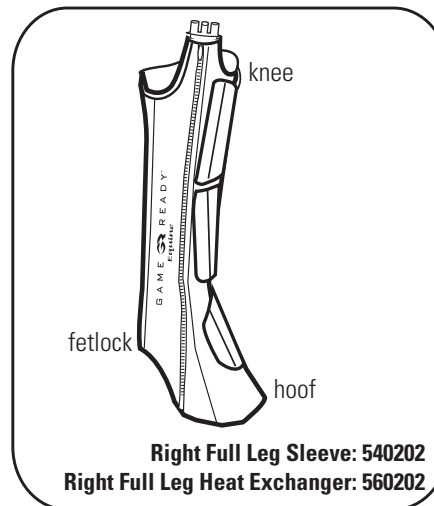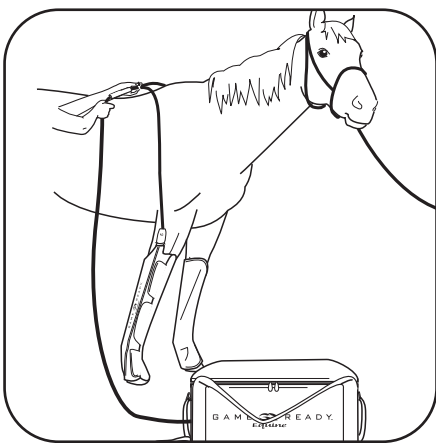

Right Full Leg Sleeve: 540202
Right Full Leg Heat Exchanger: 560202

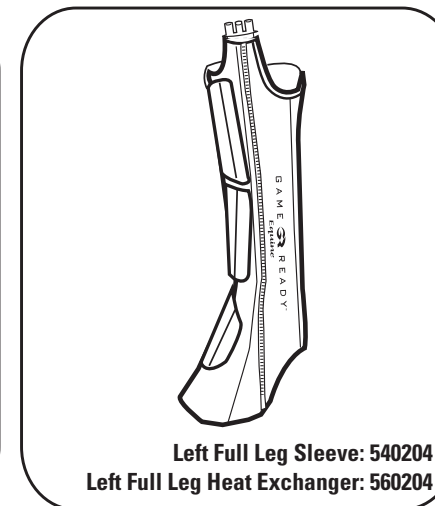

Left Full Leg Sleeve: 540204
Left Full Leg Heat Exchanger: 560204

Warning: Always consult with your veterinarian before using the Game Ready Equine System for an injury.

Note: For maximum cooling, place Wrap against bare leg.

Caution: Do not place directly against open wounds, sores, rashes infections, or stitches. First place a sterile dressing on the wound and then apply the Wrap. If pressure is uncomfortable, pause unit, and restart with lower pressure. Do not store in closed plastic bag after use, as this will promote mold growth. Do not puncture Wrap, doing so will void Game Ready Equine warranty.

Care: Gently remove Heat Exchanger from the Sleeve. Hand wash the Sleeve in cold water and mild detergent, or antibacterial soap. Hang to dry. Hand wash the Heat Exchanger, do not machine wash. Doing so will void warranty. Periodically run hydrogen peroxide through your Game Ready Equine System to keep the interior water chamber of your Wrap clean. See Control Unit User's Manual for more detail. Cold sterilization only.

Warranty Information:

Sleeve: In case of manufacturer defect, Sleeve may be returned within 7 days of purchase.

Heat Exchanger: 1 year from date of purchase. See warranty card included with Heat Exchanger.

CoolSystems, Inc.
1201 Marina Village Pkwy Suite 200
Alameda, CA 94501
1-888-GAMEREADY




Full Leg Wrap =

(left side illustrated)

Full Leg Sleeve

+

Full Leg Heat Exchanger

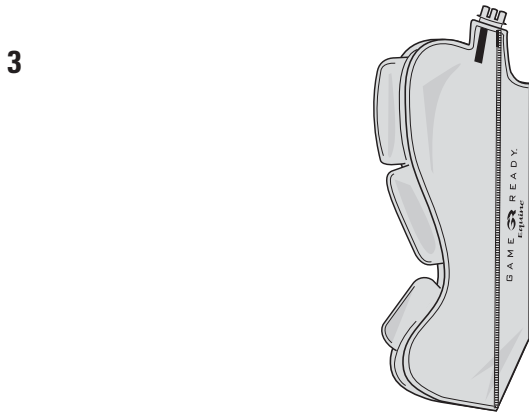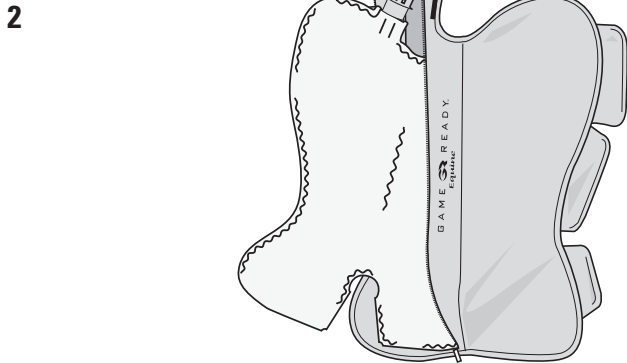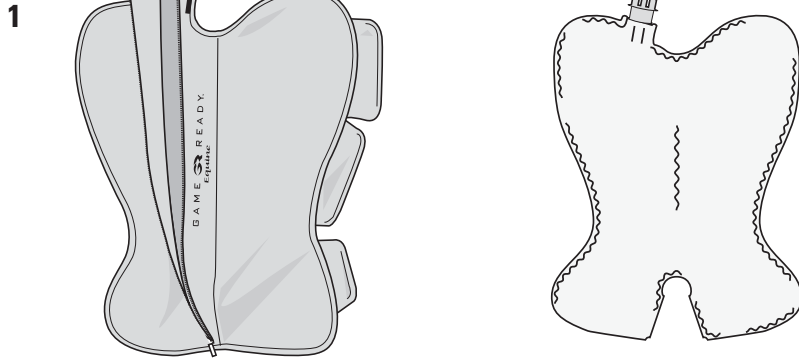

GAME READY
Equine

Right Full Leg Wrap #580202
Left Full Leg Wrap #580204

Full Leg Wrap (Sleeve/Heat Exchanger) - Insertion

fig. 1 Place Sleeve on flat surface with logo side up and open the zipper

fig. 2 Insert Heat Exchanger into the Sleeve with green side down (Green side of Heat Exchanger should touch green side of Sleeve)

fig. 3 Make sure the Heat Exchanger is flat inside the sleeve with no folds or creases. Zip up the sleeve. After assembly, fold at natural vertical fold, make sure the Wrap is flat and smooth

Full Leg Wrap (sleeve/heat exchanger) - Removal

- 1 Disconnect the Connector Hose from the Wrap
- 2 Unzip the zipper
- 3 Gently pull out the Heat Exchanger