


## Veterinary Small Utility Wrap

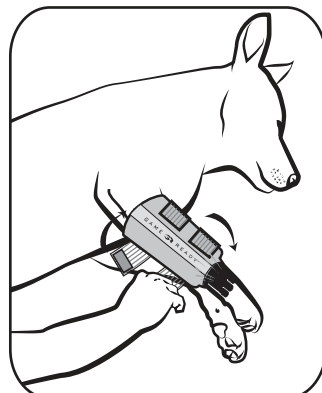


Place the Wrap(s) around the treatment site on the animal. The Veterinary Small Utility Wrap can be applied wherever swelling may occur. Secure by adjusting the elastic hook and loop closure for a snug fit.

When using one Wrap, attach the Single Connector Hose to the Wrap. Connect the opposite end of the Connector Hose into the Control Unit.

When using two Wraps simultaneously, use the Dual Connector Hose. Connect each hose end to its corresponding Wrap. Connect the single opposite end of the Connector Hose into the Control Unit.

Do not allow hose to bend the Wrap at connection. This can impede flow, and affect cooling. During treatment, the animal must be accompanied by an attendant.

After treatment, disconnect the Hose from the Wrap first, before removal of Wrap from the animal.

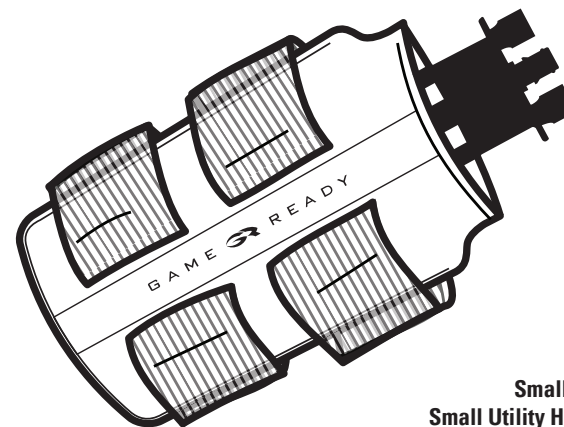

Small Utility Sleeve: 543100  
Small Utility Heat Exchanger: 563100

**Warning:** Always consult with your veterinarian before using the Game Ready System.

**Caution:** Do not place wraps directly against open wounds, sores, rashes, infections, or stitches. First place a sterile dressing on the wound and then apply the Wrap. If pressure appears uncomfortable for the animal, pause treatment, and restart with lower pressure. Do not store in closed plastic bag after use, as this will promote mold growth. Do not puncture Wrap, doing so will void the Game Ready warranty.

**Care:** Gently remove the Heat Exchanger from the Sleeve. Hand wash the Sleeve in cold water and mild detergent, or antibacterial soap. Hang to dry. Hand wash the Heat Exchanger, **do not machine wash**. Doing so will void the warranty. Periodically run isopropyl alcohol through your Game Ready System to keep the interior water chamber of your Wrap clean. See Control Unit User's Manual for more detail. Cold sterilization only.

**Warranty Information:**

**Sleeve:** In case of manufacturer defect, sleeve may be returned within 7 days of purchase.

**Heat Exchanger:** 1 year from date of purchase. See warranty documentation included with Heat Exchanger.

 CoolSystems®, Inc.  
1800 Sutter Street, Suite 500  
Concord, CA 94520  
1-888-GAMEREDY  
1-510-868-2100  
[www.gamereadyveterinary.com](http://www.gamereadyveterinary.com)

## Veterinary Small Utility Wrap =

Utility Sleeve + Utility Heat Exchanger

Fig 1

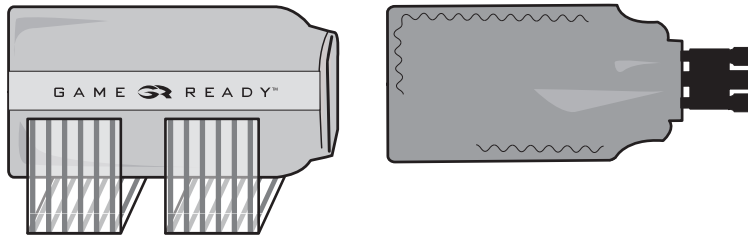


Fig 2


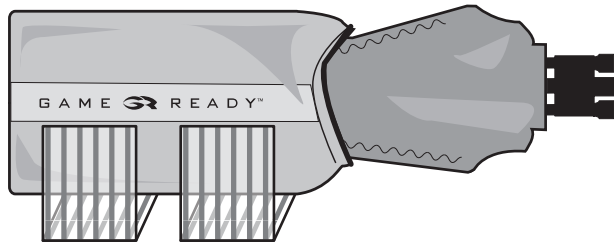


Fig 3

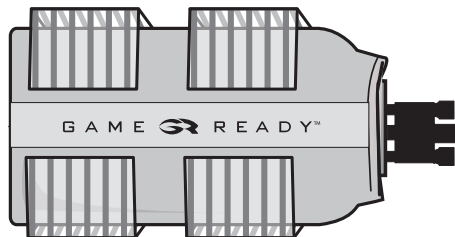

### Veterinary Small Utility Wrap (Sleeve/Heat Exchanger) - Insertion

**fig. 1** Place Sleeve on flat surface with logo side up

**fig. 2** Insert Heat Exchanger into the Sleeve with blue side down  
(Blue side of Heat Exchanger should touch blue side of Sleeve)

**fig. 3** Make sure the Heat Exchanger is flat inside the Sleeve with no folds or creases. After assembly, lay the blue side up, make sure the Wrap is flat and smooth

### Veterinary Small Utility Wrap (Sleeve/Heat Exchanger) - Removal

- 1 Disconnect the Connector Hose from the Wrap
- 2 Gently pull out the Heat Exchanger